

Medichill Dental/Facial Cool Cubes and Cover



AUSTRALIAN MADE
Medichill Cool Cubes
with adjustable
compression Head Cover
 Simple and effective
 pain management



- LIGHTWEIGHT
- FLEXIBLE
- DISPOSABLE
- LATEX FREE

By combining the unique Medichill Cool Cubes with an adjustable cover, you are now able to offer pain relief and comfort for varieties of head and jaw trauma as well as dental extractions and reconstructions. The benefits are numerous, from minimising cross-infection, ease and speed of application and above all patient comfort. Ideal for head knocks, toothache and jaw pain.

Rated between 9-10 out of 10 on hospital evaluation trials.

DENTAL PAD AND COVER

Code	Pack Qty	Price	Unit Cost
DPC-10	10	45.00	4.50
DPC-20	20	82.00	4.10
DPC-50	50	190.00	3.80
DPC-100	100	350.00	3.50
DPC-200	200	608.00	3.04
Larger quantities available		P.O.A	

COVERS ONLY

DC-10	10	35.00	3.50
DC-20	20	64.00	3.20
DC-50	50	147.50	2.95
DC-100	100	270.00	2.70
DC-200	200	480.00	2.40
Larger quantities available		P.O.A	

Prices are plus GST and Delivery

113/396 Scarborough Beach Rd
Osborne Park WA 6017

Freecall 1800 35 1232

Fax 08 9444 2509

medichill.com.au

info@medichill.com.au

medichill.australia




Medichill Ice Pads are manufactured to ISO 9001

Like us on Facebook
facebook.com/medichill

How to use the Medichill Dental Pad and Cover




- 1 Cut sheet in half between cells as indicated. Our research indicates that this sheet size provides maximum flexibility and cool contact when applied.

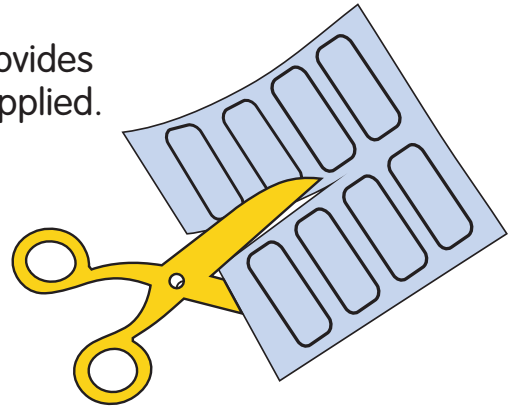

- 2 Soak the sheets in clean water until cells are fully hydrated and firm to touch. Wipe off any excess moisture, and place in freezer. You can place in a zip lock bag to aid cleanliness when freezing or if the sheets are to be stored for a longer period of time.

- 3 Remove from freezer and place inside the cover pockets, one to each side. Apply the cover to patients head, securing with velcro and adjust for patient comfort. The pocket side of the cover goes against the skin.

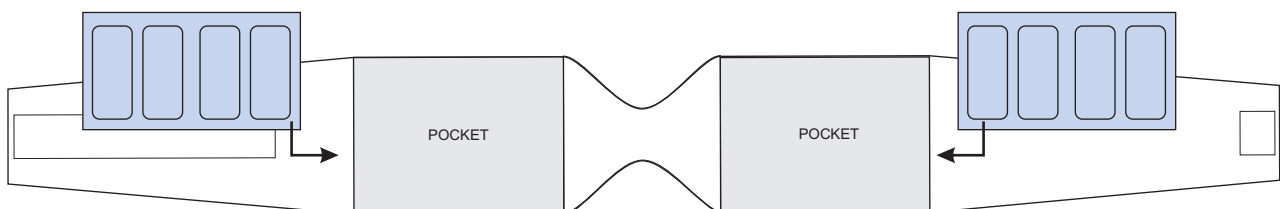

- 4 Same patient can re-use until pain subsides. Maximum time per application 20 minutes, then break for 20 mins and then re-apply for as long as required. Natural dehydration of the pads occurs over time, simply soak the sheets again to re-hydrate and re-commence the process.